NOBIL-3D

CS02

3D Printer User Manual

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V3.0

Dear Consumers,

Thank you for choosing our products. For the best experience, please read the instructions before operating the Printer. Our teams will always be ready to render you the best services. Please contact us via the phone number or e-mail address provided at the end when you encounter any problem with the Printer.

For a better experience in using our product, you can also learn how to use the printer in the following ways:

View the accompanied instructions and videos in the U disk.

Visit our official website www.hugedental.com to find relevant software/hardware information, contact details and operation and maintenance instructions.

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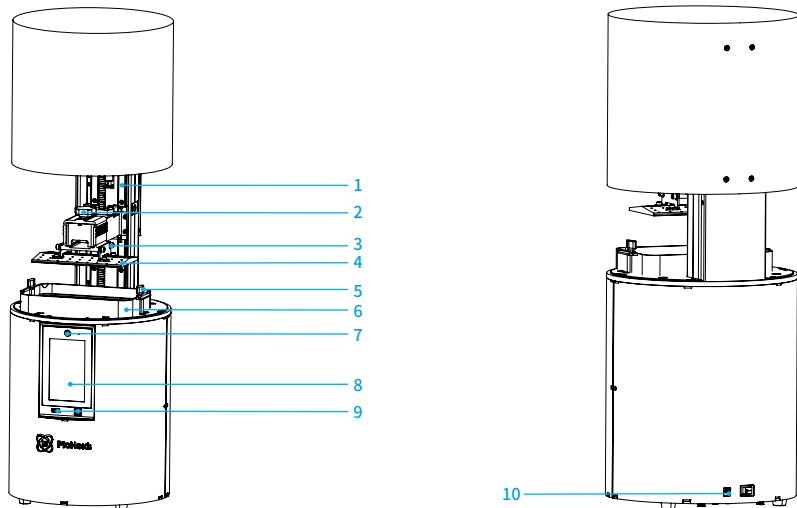
- ① Do not use the printer any way other than described herein in order to avoid personal injury or property damage.
- ② Do not place the printer near any heat source or flammable or explosive objects. We suggest placing it in a well-ventilated, low-dust environment.
- ③ Do not expose the printer to violent vibration or any unstable environment, as this may cause poor print quality.
- ④ Please use recommended resin to in case damage of machine.
- ⑤ Do not use any other power cable except the one supplied. Always use a grounded three-prong power outlet.
- ⑥ Please do not open the plastic cover during usage, otherwise the printing will be interrupted.
- ⑦ Do not wear cotton gloves when operating the printer. Such cloths may become tangled in the printers moving parts leading to burns, possible bodily injury, or printer damage.
- ⑧ Please wait a moment after the print is finished. And please wear gloves to take out the print by tools.
- ⑨ Clean the printer frequently. Always turn the power off when cleaning, and wipe with a dry cloth to remove dust, adhered printing plastics or any other material off the frame, guide rails, or wheels. Use glass cleaner or isopropyl alcohol to clean the print surface.
- ⑩ Children under 10 years should not use the printer without supervision.
- ⑪ This machine is equipped with a security protection mechanism. Do not manually move the nozzle or printing platform mechanism manually while booting up, otherwise the device will automatically power off for safety.
- ⑫ Users should comply with the laws and regulations of the corresponding countries and regions where the equipment is located (used), abide by professional ethics, pay attention to safety obligations, and strictly prohibit the use of our products or equipment for any illegal purposes. Piocreat will not be responsible for any violators' legal liability under any circumstance.

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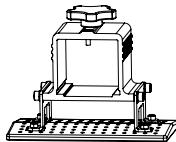
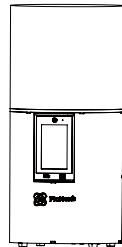
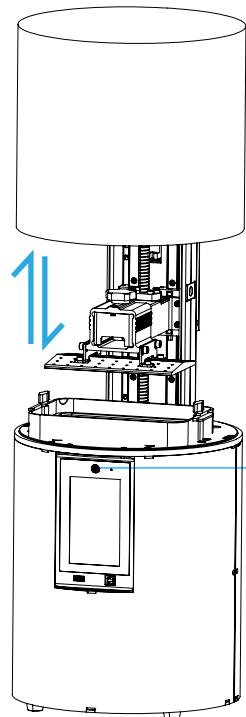
⑧ Power cord x 1
电源线

⑨ Disposable latex
gloves x 1
一次性乳胶手套

Tips: the above accessories are for reference only, in kind prevail! 注意:以上配件仅供参考,请以实物为准!

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Open state | 开盖状态

After taking out the device from the packing box, connect the device to the power supply and wait for it to start. After the boot interface is displayed, click the thumb silkscreen on the top of the touch screen with your finger. When you hear a beep, the upper cover of the device will rise (for example, when the upper cover goes down for the first time it is the zeroing operation. After the upper cover is reset to zero, click the thumb silkscreen again, and the upper cover will rise).

Movement rules of the upper cover: when the bottom limit switch detects that the limit sheet has not returned to zero, the priority is to return to zero, that is, the upper cover goes down, and when the limit sheet touches the limit switch, it is determined that the zero return is successful, and click the thumb silkscreen again, the upper cover runs upwards.

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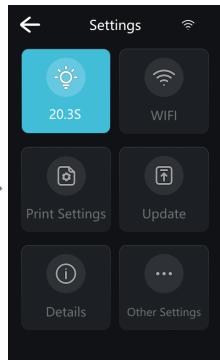
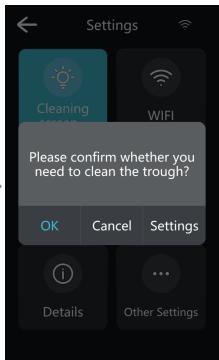
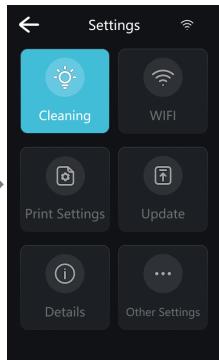
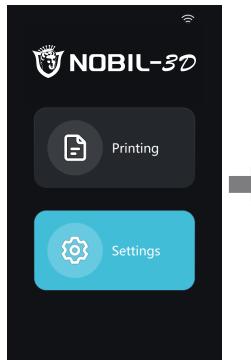
Note:

1. During the rising and falling process of the upper cover, click the thumb silkscreen upper cover movement to pause, click the thumb silkscreen again, and the movement will continue.
2. In process of the upper cover, long press the thumb silkscreen and the upper cover will be forced to move zero (that is down movement)
3. Please ensure to close the upper cover before printing, and please don't open the upper cover during printing and avoid being layered patterns on the printed model.

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5. Interface Information 界面信息

Screen clearing: material tank cleaning;

Z-axis movement: Back to Zero and leveling;

Print settings: printing parameters, device control, feeding system and Pre-Heat;

System settings: set WALN, set language and region, upgrade firmware version (local upgrade & network upgrade); view device information, restore factory settings.

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

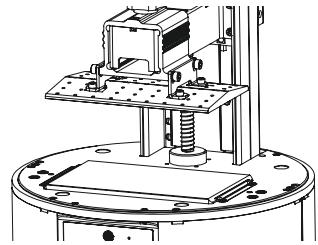
6. Level Platform 调平平台

If the platform does not stick to the print screen, do as the following:

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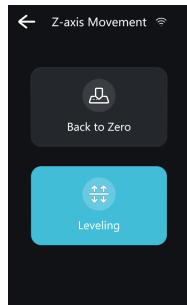
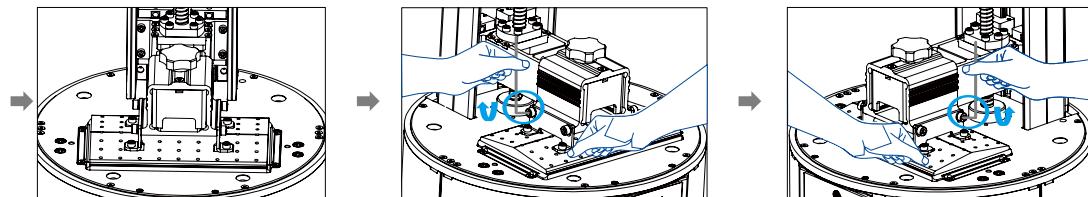
1. After unscrewing the leveling screws, press "Leveling" to keep the platform and the print screen parallel.

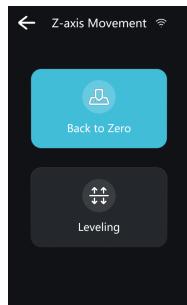2. Gently press the platform and tighten the 4 screws clockwise and diagonally, and the platform will be successfully leveled.

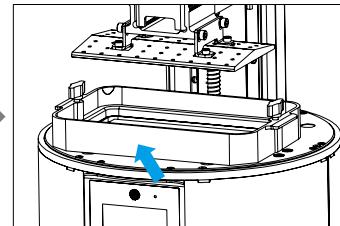
3. Press "Auto-homing", move up the z-axis to a proper position, and then tighten the fixing screws at both ends of the resin vat.


6. Level Platform 调平平台

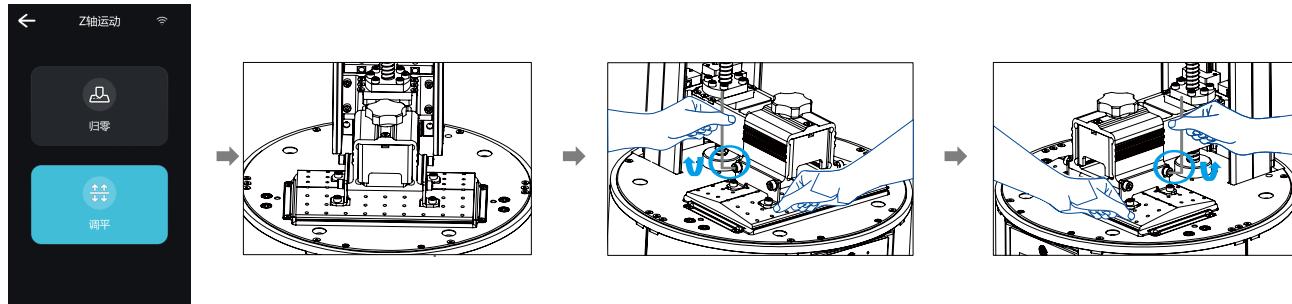If the platform does not stick to the print screen, do as the following:
如果没有贴紧打印屏,请进行如下操作:

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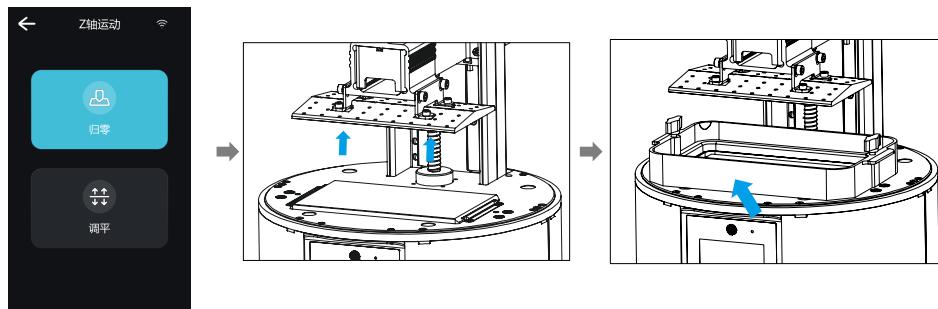
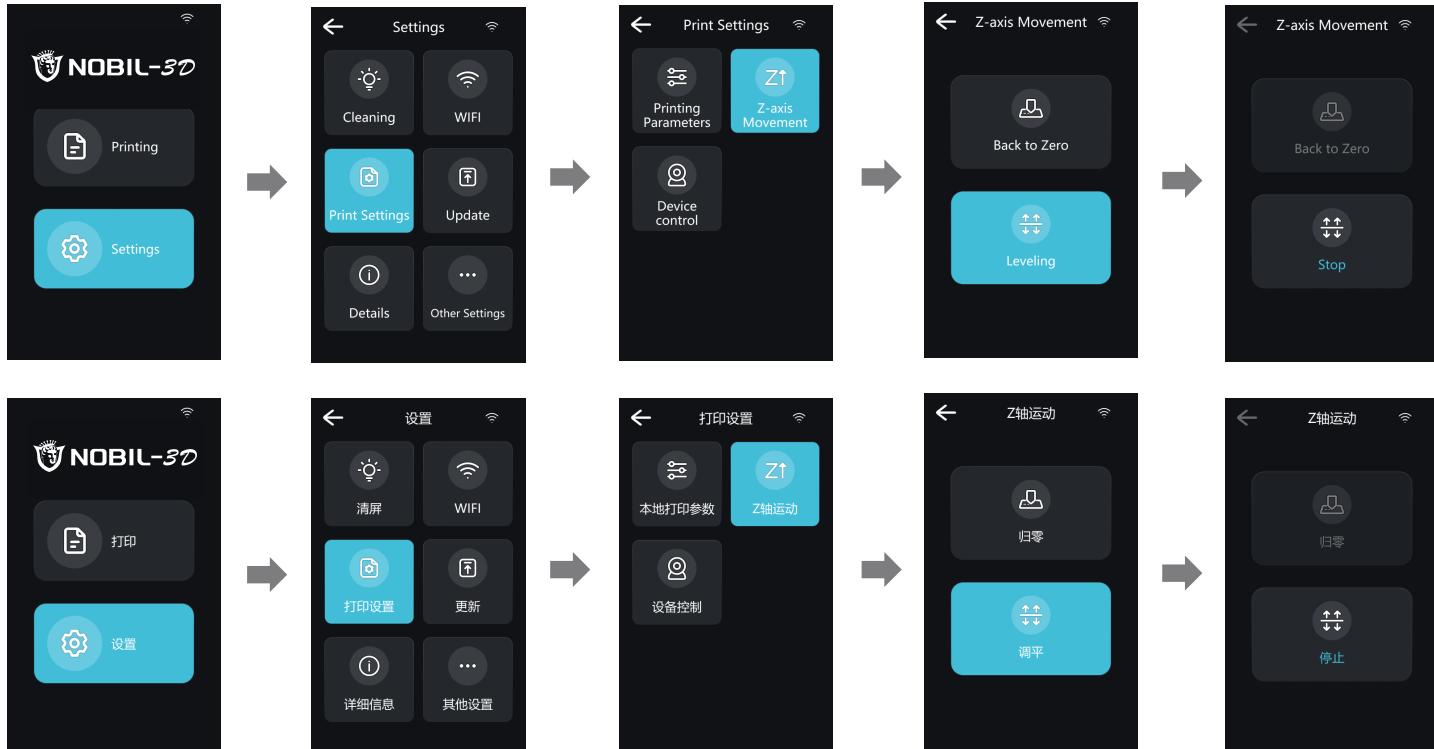
7. Load Resin 装料

Pour the UV photosensitive resin into the resin vat. The tank capacity scale is 400ml.

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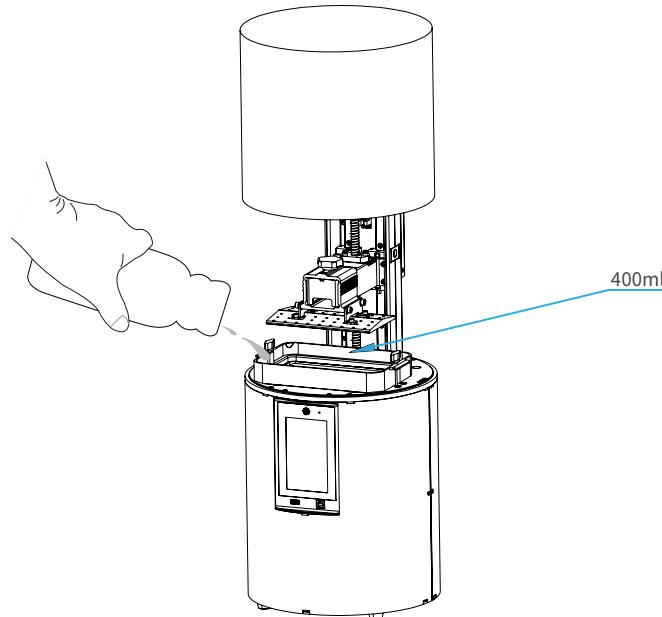Tips: How to change resin:

Take out the resin vat slowly, pour out the remaining resin into a container, clean the resin vat, pour in new resin, and then install the resin vat to the machine.

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Caution : Resin may cause skin irritation or an allergic skin reaction. Please wear gloves before use to avoid direct contact with the skin.

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

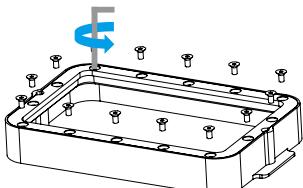
b.

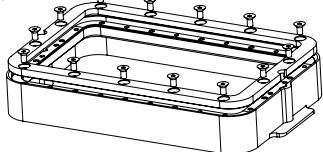c.

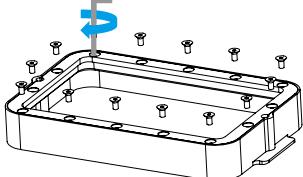- a. First, remove the 14 screws at the bottom of the vat and take down the old release film.

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- b. Prepare the new release film, peel off the protective films on both sides of the release film.

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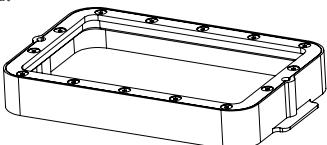
- c. Place the clamping, install 14 countersunk head screws and use the opposite angle to tighten.

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- d. The installed release film is as shown.
安装好的离型膜如下图。

d.

Parameters

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尺寸:216x158mm
(Size)

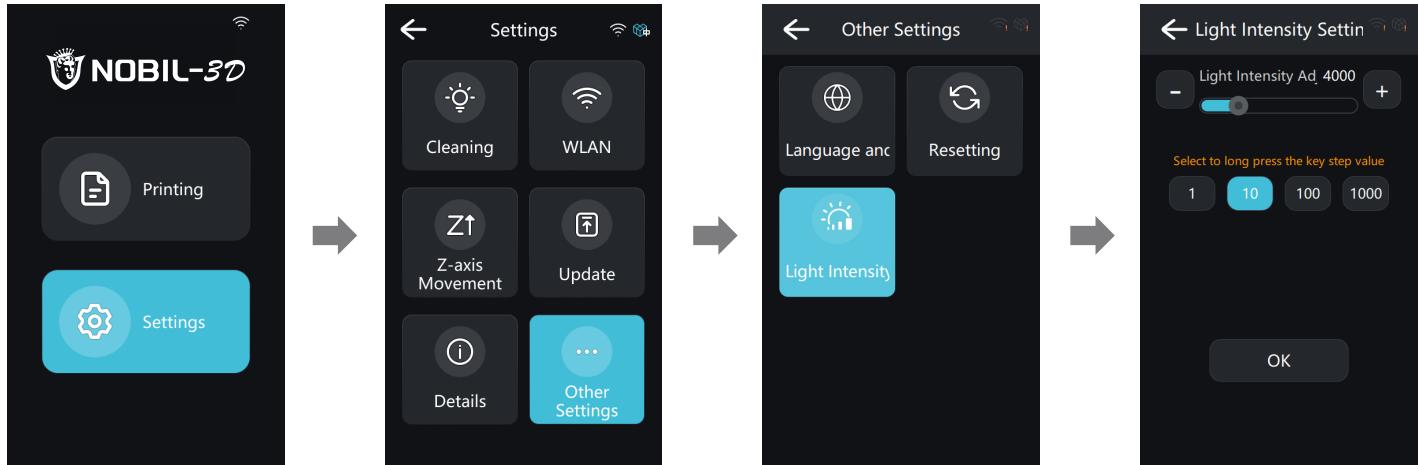
厚度:0.15-0.2mm
(Thickness)

透光率:95%
(Light Transmittance)

9. Light intensity setting 光强设置

Warning: The light intensity of the machine has been adjusted before leaving the factory. Generally, it does not need to be adjusted. After the machine has been used for a long time, if it is found that the light intensity does not meet the needs of use, it can be adjusted by itself.

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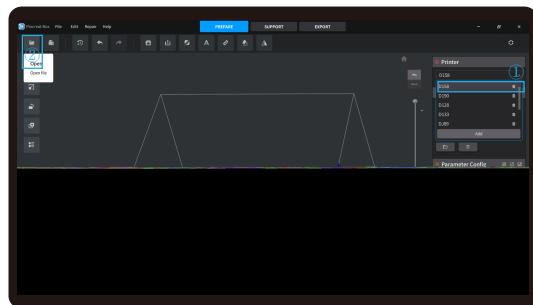
1. Slice


HUGE STUDIO

1. Open the HUGE STUDIO slicing software.

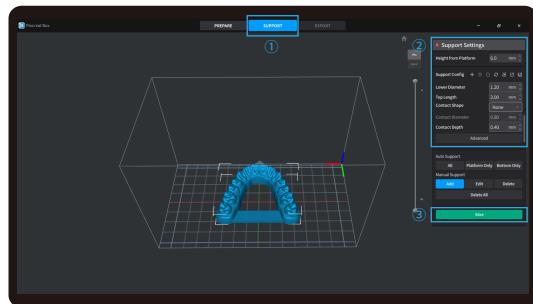
- ① Click the right printer column to select the device model
- ② And click "OPEN" on the top left of the interface to import the model file;

- #### 3. Add bracket to the model (skip steps ① and ② if no bracket is added).
- ① Click the icon above → ② Set the corresponding parameters
 - ③ Click the slice;

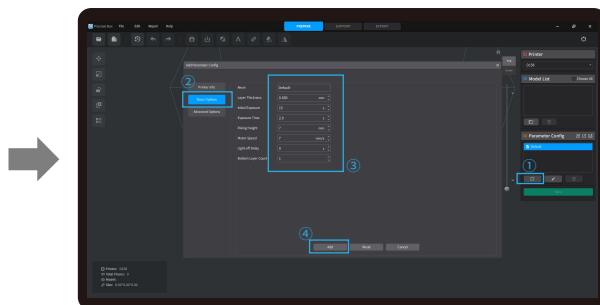
Note: The published UI of slicing software shall prevail for the interfaces of HUGE STUDIO.

2. Setting Printing Parameters (If use Default-Parameters,skip this step)

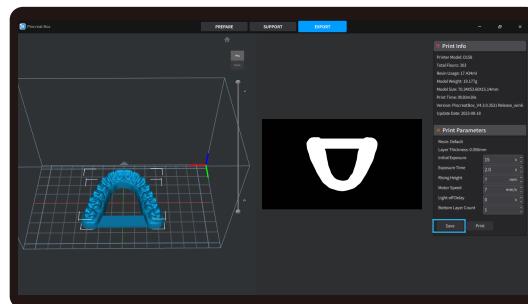- ① Click the "Add" icon in the Parameter Configuration
- ② Click the Basic Setting Option
- ③ Set the appropriate parameters → ④ Click "Add"

- #### 4. Waiting for the auto slicing to complete (you can adjust the print parameters as required).
- Click "Save" icon to save the sliced file.

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
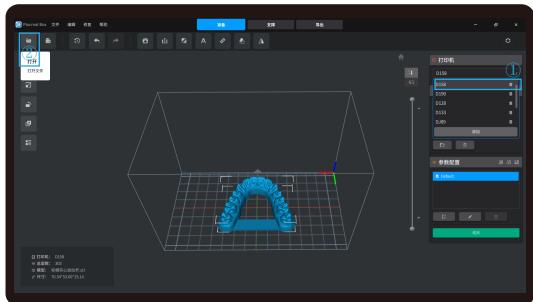
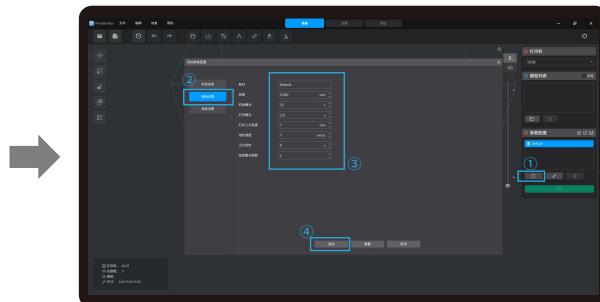
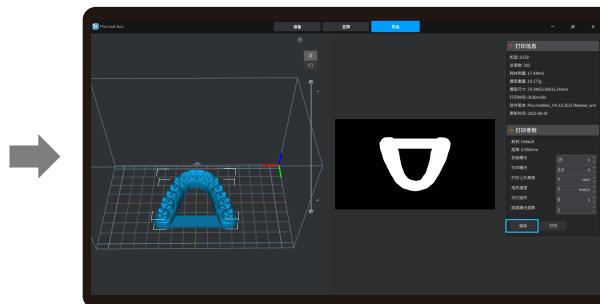
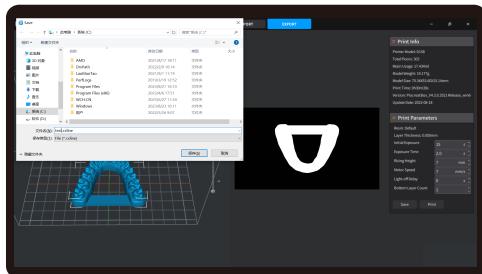
2. Print

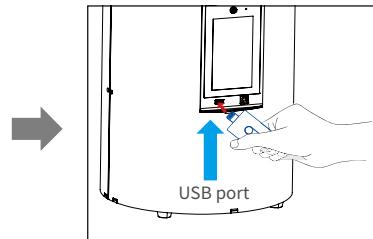
1. Generate .cxline file, and save the file to U disk.

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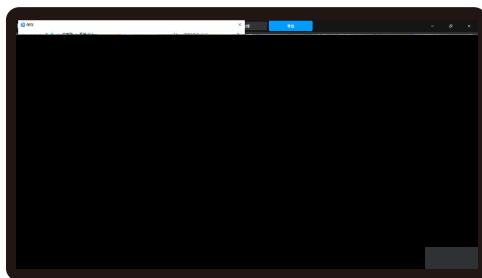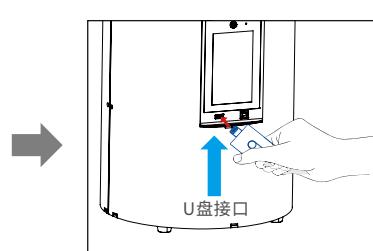
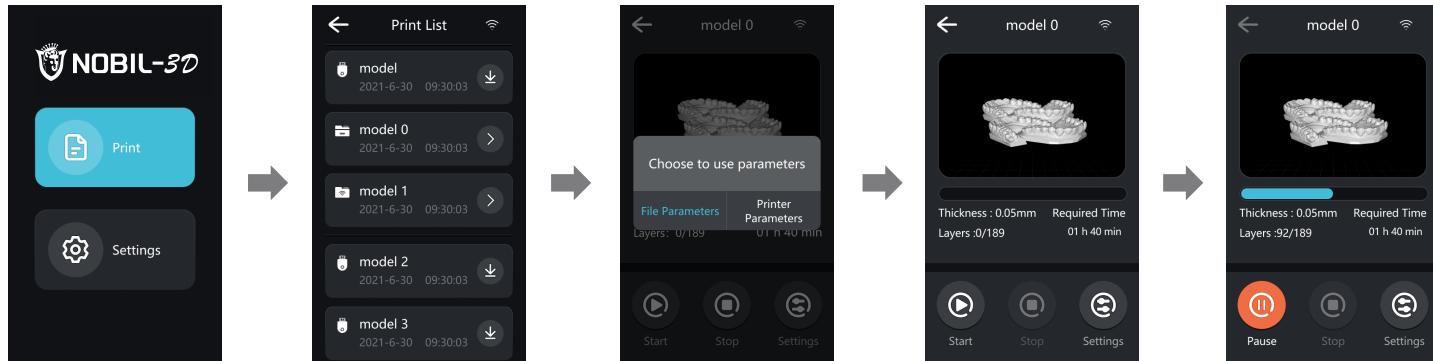
Warning: During the printing process the machine can't be shaken, so as not to avoid influencing affect the printing quality.
警告：在打印过程中，机器不能摇动，以免影响打印质量。

10.USB Disk Printing U盘打印


11. WIFI Print 无线打印

>>>


Slice


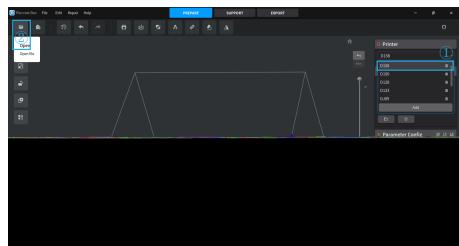
HUGE STUDIO

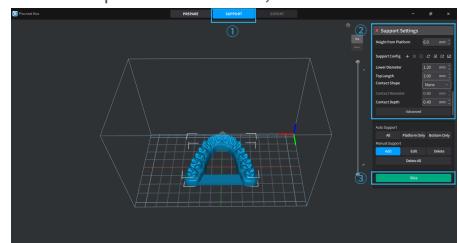
1. Open the HUGE STUDIO slicing software.
→①Click the right printer column to select the device model
→②And click "OPEN" on the top left of the interface to import the model file;

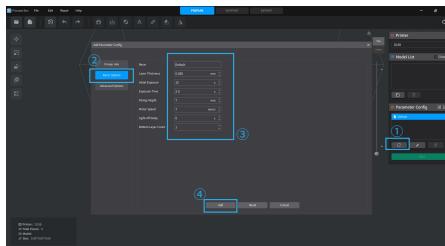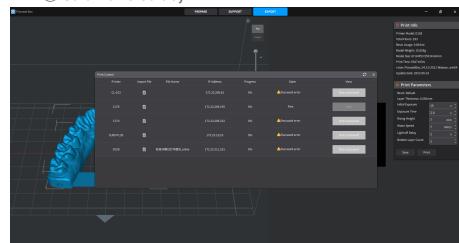3. Add bracket to the model (skip steps ① and ② if no bracket is added).
→①Click the icon above →②Set the corresponding parameters
→③Click the slice;

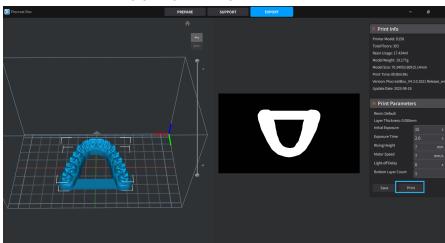2. Setting Printing Parameters (If use Default-Parameters,skip this step)
→①Click the "Add" icon in the Parameter Configuration
→②Click the Basic Setting Option
→③Set the appropriate parameters →④Click "Add"

4. Waiting for the auto slicing to complete (you can adjust the print parameters as required).
→Click "Print" icon to use WIFI-printing (* slicing tutorial see the official website slicing video www.hugedental.com)

5. Select the file on the machine to initiate printing when the transfer is complete.

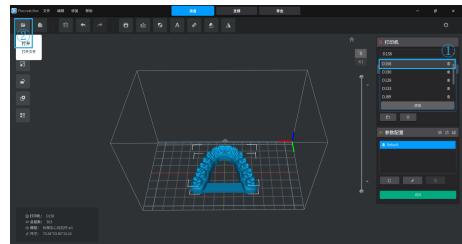
Note: Make sure the HUGE STUDIO and the printer are connected to the same WiFi.

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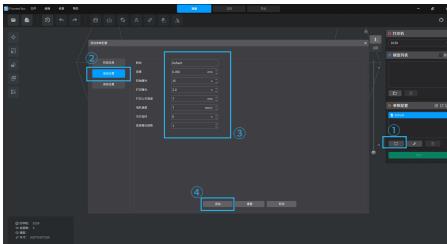
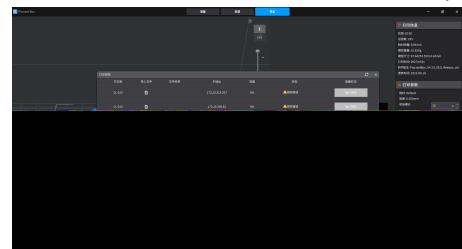
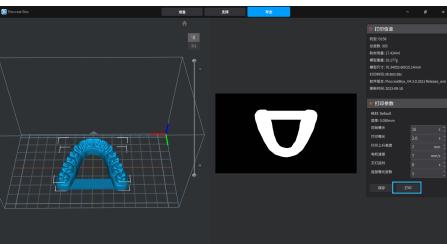
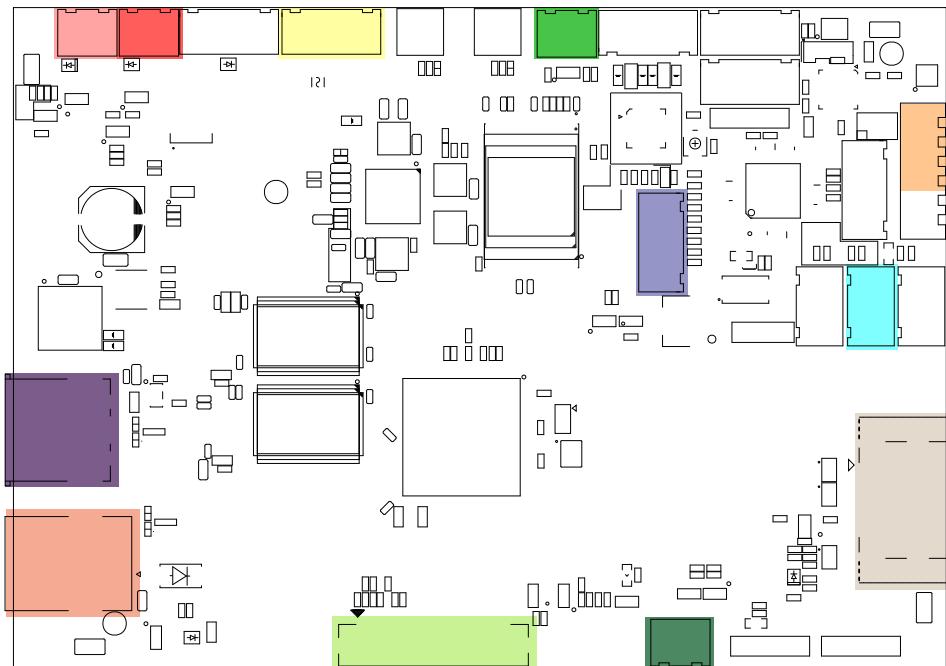
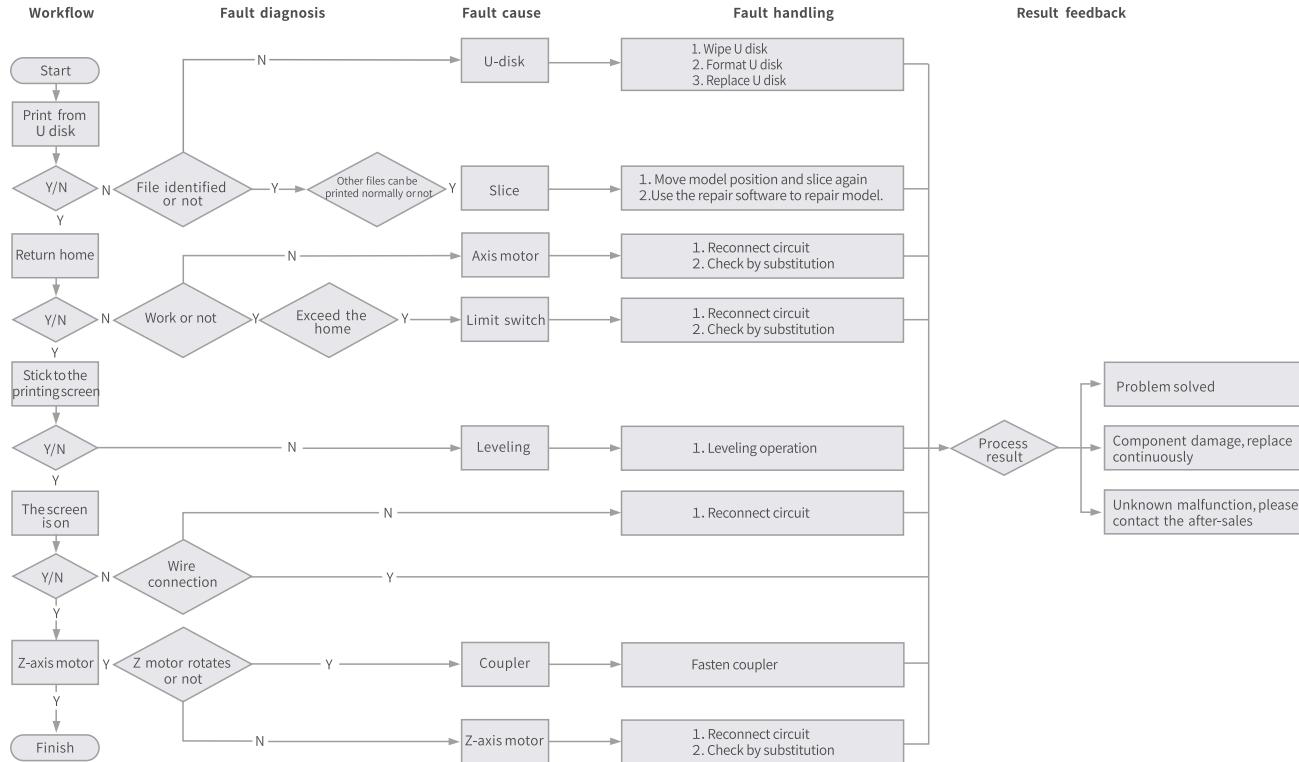
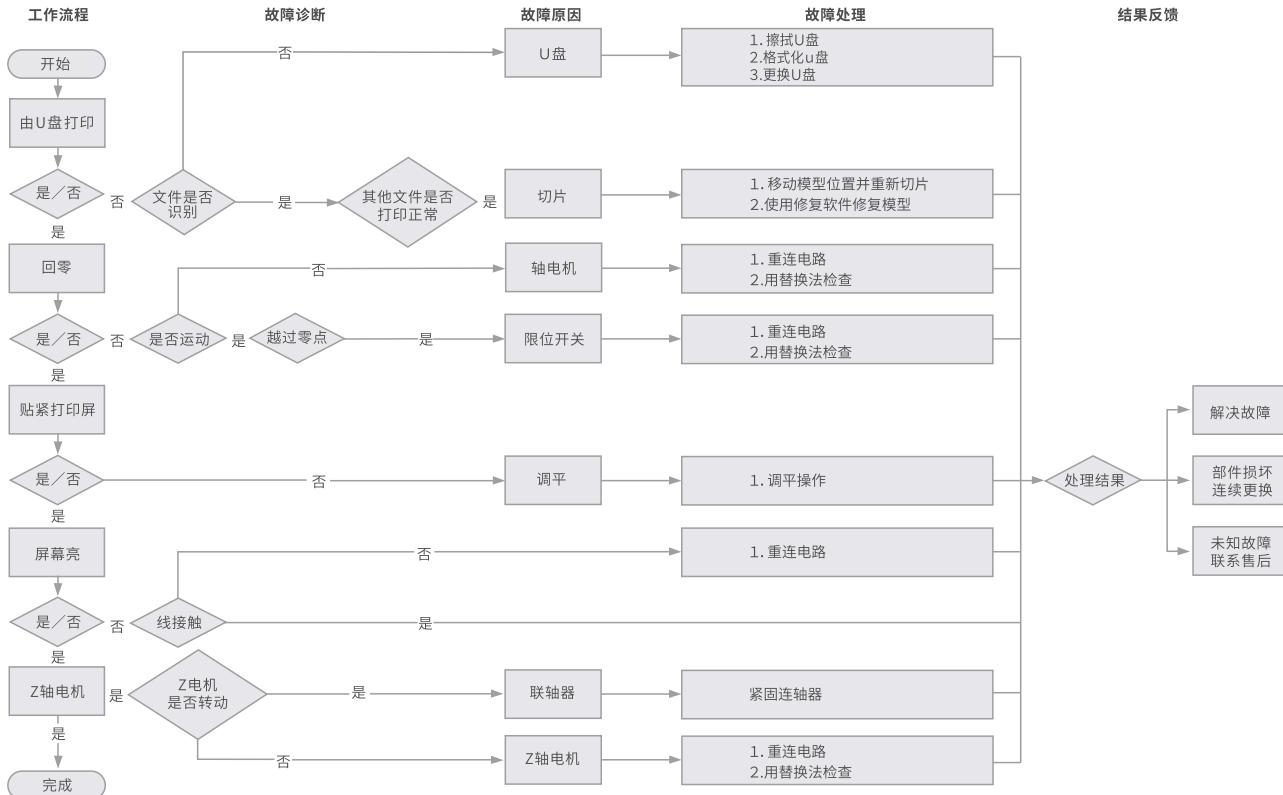
Scan the code to know more

Warranty Policy ←

- 1-year Limited Warranty since the date of product purchase.
- No Warranty for the Release Film and Product Gift (U disk, Gloves, Screwdriver, Paper funnel, Brush, etc.)

What Is Covered? ←

Any defects in material, workmanship of product will be covered until you provide any relating documents and information, including but not limited to:

- User of the product are in the country of purchase
- Defective product is a **HUGE** product
- Proof of Purchase

What Is Not Covered? ←

PIOCREAT does not guarantee the service as a result of:

1. The proof of purchase being altered or made illegible.
2. The product label or serial number on the product being altered or made illegible.
3. Normal wear of accessories or consumables for use as release film.
4. Malfunction or damage caused by personal retrofitting, or improper installation and usage.
5. Malfunction or damage caused by use in non-recommended working environments.
6. Malfunction or damage caused by overuse (overload) or wrong maintenance (damp, mildew, etc.)
7. Malfunction or damage caused by Force Majeure (fire hazard, earthquake, lightning and floods, etc.)
8. Malfunction or damage caused by the use of other brand parts or consumable.

Technical Support ←

Please enjoy our technical support of the whole lifetime (Except technical problem beyond product itself) after purchase the product while product documents as user manual attached in U disk for instruction.

Please provide product Serial Number when apply for technical support.

Kindly Notification ←

Please read the user manual and warranty card carefully before the use. For any service, Please priorly contact local seller or contact us via Email: marketing@hugedental.com for support. See www.hugedental.com for specific warranty information about your product and procedures to act on a warranty claim.

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WARRANTY

Before returning the product and filling in a warranty, please contact after-sale person for going through after-sale formality. And attach this waranty card along with the returned machine.

Repair

Change

Return

Name: _____ Telephone: _____

Address: _____

Malfunction And Damage Depiction Or Return And Change Reasons\\ Suggestions:

Serial Number: _____ Order Number: _____

Channel: Platform Offline

Date of purchase Day _____ Mon. _____ Year _____

Repair Records:

Note: Client need filling in basic infor. and return reasons. Repair records shall retain for technicians.

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Notes 笔记

Notes 笔记

Due to the differences between different machine models, the physical objects and the final images can differ. The final explanation rights shall be reserved by Shandong Huge Dental Material Corporation
因每款机型不同，实物与图可能有所差异，请以实物为准，最终解释归山东沪鸽口腔材料股份有限公司所有。

Scan To Learn More
扫码了解更多

Shandong Huge Dental Material Corporation

Add: No.68 Shanhai Road, Donggang District, Rizhao City, Shandong Province,
276800, P.R.China

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